



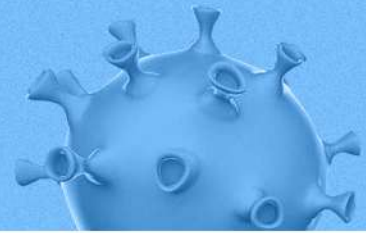


COVID-19 Antigen Rapid Test Cassette

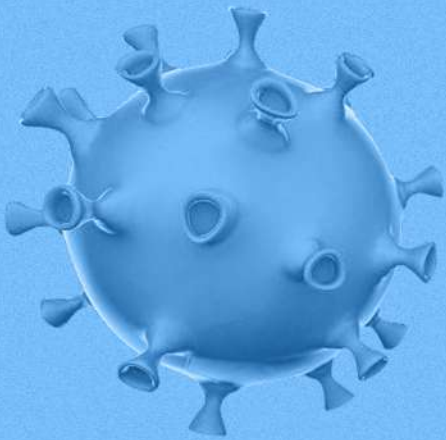

Hangzhou Clongene Biotech Co., Ltd.
en.clongene.com

COVID-19 & SARS-CoV-2

COVID-19 is an acute respiratory infectious disease caused by novel coronavirus (SARS-CoV-2), and people are generally susceptible. Based on the current epidemiological investigation, the incubation period is 1 to 14 days, mostly 3 to 7 days. Novel coronavirus includes four typical structural proteins: Spike Protein, Envelope Protein, Membrane Protein and Nucleocapsid Protein.

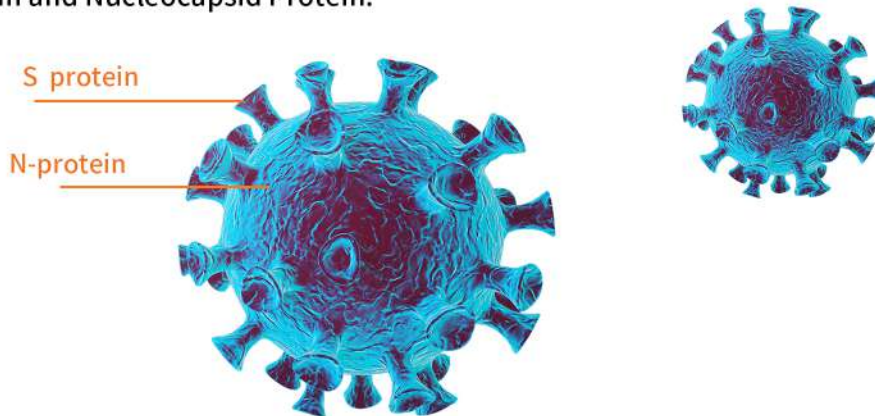

Nucleocapsid (N) protein is the most abundant protein with highly conserved in SARS-CoV-2. N protein is used as the core raw material of rapid diagnostic reagent for immunology in the market.

Clongene has developed the COVID-19 Antigen Rapid Test Cassette. The COVID-19 Antigen Rapid Test is a lateral flow immunoassay intended for the qualitative detection SARS-CoV-2 nucleocapsid antigens in nasopharyngeal swab and oropharyngeal swab from individuals who are suspected of COVID-19 by their healthcare provider.



Kit Contents

Package	Test Cassette	Extraction reagent	Sterilized Swab	Extraction Tube & Dropper Tip
				

Product Features



CE certification



Instant result at 15 minutes



Easy to collect samples



Results are clearly visible




No equipment required



Suitable for large-scale rapid screening

Principle

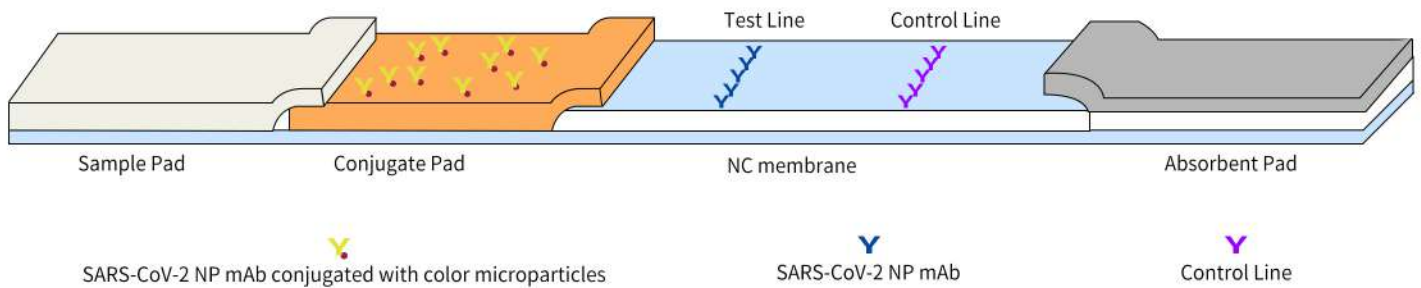


The COVID-19 Antigen Rapid Test is a lateral flow immunoassay based on the principle of the double-antibody sandwich technique. If the specimen contains SARS-CoV-2 antigen, a colored test line (T) would be visible in the result window. Absence of the T line suggests a negative result. The control line (C) is used for procedural control, and should always appear if the test procedure is performed properly.


Specimens

The detect specimens include nasopharyngeal swab and oropharyngeal swab.

Nasopharyngeal swab

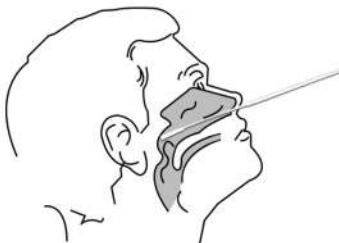

Oropharyngeal swab

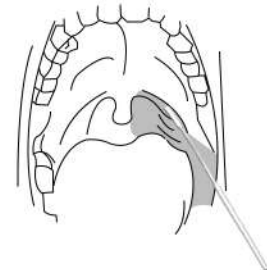

Inadequate specimen collection, improper specimen handling and/or transport may yield false results; therefore, training in specimen collection is highly recommended due to the importance of specimen quality to obtain accurate test results.

Test Procedure

Take nasopharyngeal swab for example.


1

Add all of the extraction reagent into an extraction tube.




2

Tilt patient's head back about 70°. Insert the swab through the nostril parallel to the palate until resistance is encountered or the distance is equivalent to that from the ear to the nostril of the patient

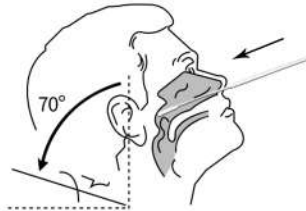


3

Gently rub and roll the swab, and leave swab in place for several seconds to absorb secretions.

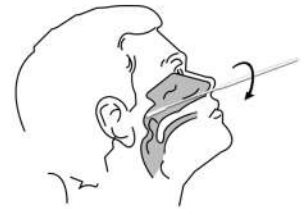


4

Slowly remove swab while rotating it.

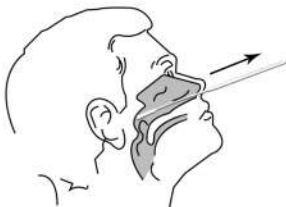


5

Insert the swab specimen into the extraction tube. Roll the swab at least 5 times and leave the swab in the extraction tube for one minute.

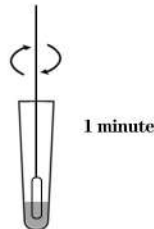


6

Remove the swab while squeezing the sides of the tube to extract the liquid from the swab.

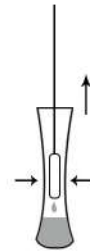


7

Cover the extraction tube with a dropper tip tightly.

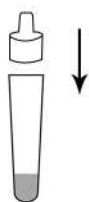


8

Transfer 3 drops (approximately 100µL) to the specimen well of the test cassette.

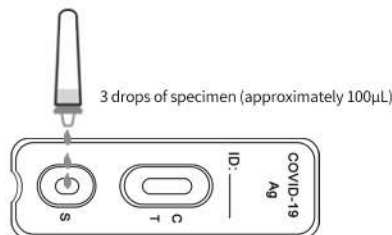


9

Interpret the test results at 15 minutes. Do not read results after 20 minutes.

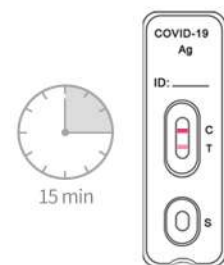

Interpretation of Results

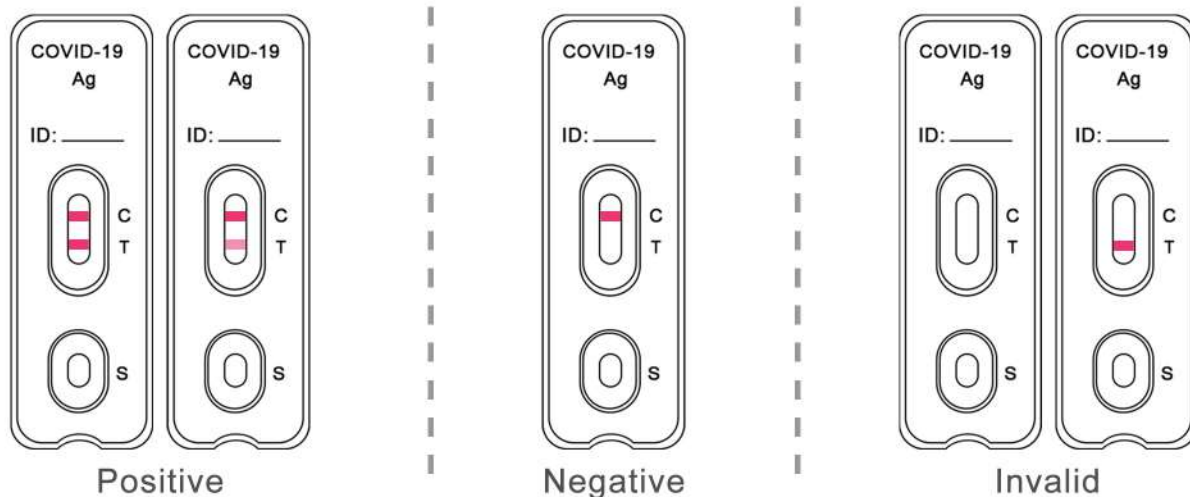

Performance Characteristics

Clinical Performance

770 nasopharyngeal swabs were collected from individual symptomatic patients. The swabs were detected by COVID-19 Antigen Rapid Test of Clongene and the RT-PCR. Summary data was showed as below:

COVID-19 Antigen		RT-PCR (Ct value ≤ 33)		Total
		Positive	Negative	
CLUNGENE [®]	Positive	145	2	147
	Negative	3	593	596
Total		148	595	743

PPA (Ct ≤ 33): 98.0% (145/148), (95%CI: 94.2% ~ 99.3%)

NPA: 99.7% (593/595), (95%CI: 98.8% ~ 99.9%)

COVID-19 Antigen		RT-PCR (Ct value ≤ 37)		Total
		Positive	Negative	
CLUNGENE®	Positive	161	2	163
	Negative	14	593	607
Total		175	595	770

PPA (Ct ≤ 37): 92.0% (161/175), (95%CI: 87.0% ~ 95.2%)

NPA: 99.7% (593/595), (95%CI: 98.8% ~ 99.9%)

- PPA - Positive Percent Agreement (Sensitivity)
- NPA - Negative Percent Agreement (Specificity)

Limit of Detection (Analytical Sensitivity)

The study used cultured SARS-CoV-2 virus, which is heat inactivated and spiked into nasopharyngeal swab specimen. The Limit of Detection (LoD) is 5.7×10^2 TCID₅₀/mL.

Cross Reactivity (Analytical Specificity)

We have evaluated 32 commensal and pathogenic microorganisms that may be present in the nasal cavity and no cross-reactivity was observed.

Interference

17 potentially interfering substances with different concentration were evaluated and found no affect to the test performance.

High-dose Hook Effect

The COVID-19 Antigen Rapid Test was tested up to $1.0 \times 10^{5.67}$ TCID₅₀/mL of inactivated SARS-CoV-2 and no high-dose hook effect was observed.



Hangzhou Clongene Biotech Co., Ltd.

Add: No.1 Yichuang Road, Yuhang Sub-district, Yuhang District, 311121 Hangzhou, China

Tel: +86 (0)571-88262120

Fax: +86 (0)571-88261752

E-mail: marketing@clongene.com

